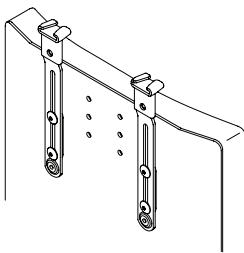
Shoulder Harness Strap Guides

INSTALLATION AND USER'S INSTRUCTIONS

ART NOS. HW100-150, HW100-200, HW105S, HW105L

These instructions provide important information for the safe use and maintenance of all Bodypoint Shoulder Harness Strap Guides. Give these instructions to the user or their caregiver and review them to ensure that they are understood.

⚠ WARNING! Product should be installed and fitted by a qualified rehab technician.

This product is intended for mounting positioning harnesses. It is NOT intended for use as a transportation safety device.

⚠ WARNING! This product should only be used for positioning a person in a wheelchair or work chair. It is NOT intended for use as a transportation safety device, as a personal restraint device, or in any other application where its failure could result in injury. Misuse of this product is unauthorized and unsafe.

⚠ WARNING! If a serious incident occurs related to the use of this product, it should be reported to the manufacturer (Bodypoint, Inc.) and the local Competent Authority.

PERIODIC SAFETY AND PERFORMANCE

CHECKS: To ensure user safety, this product must

be checked periodically for function and signs of wear. If the product does not function correctly or if significant wear is found, stop using it and contact your supplier for qualified repair or replacement by Bodypoint. Under no circumstance should this product be altered or repaired by unqualified persons — health and safety depend on it!

SCRAP/DISPOSAL: Product is made of materials which can be disposed of safely without special precautions at the end of its useful life.

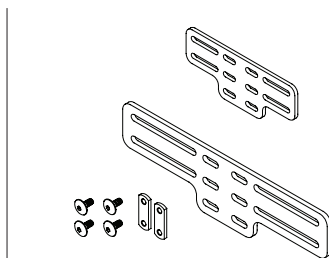
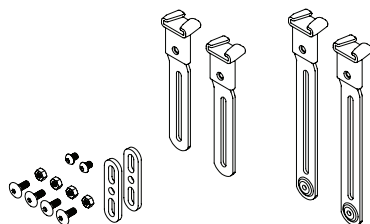
WARRANTY: This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

For more information on Bodypoint products, and a list of distributors outside the USA go to www.bodypoint.com.

PACKAGE CONTENTS

Package may contain one or more of the following:

- Strap Guides, 150mm or 200mm long, with fasteners
- Mounting Adapter, Small (185mm) or Large (270mm wide), with fasteners

BP Bodypoint®


Bodypoint Europe BV
Kerkstraat 29
7396 PD Terwolde
The Netherlands

Medical
Device

558 First Ave. S., Suite 300 | Seattle, WA 98104 | 206.405.4555 | 800.547.5716 | www.bodypoint.com

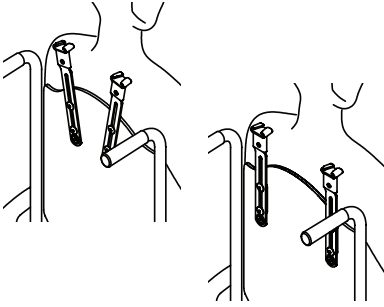INSTALLATION REQUIREMENTS

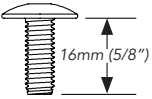
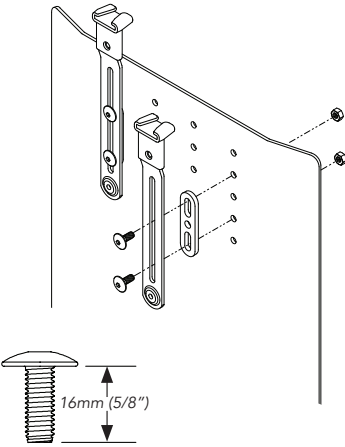
Wheelchair with a solid backrest of metal or plastic, up to 4mm (5/32") thick. Thicker backrests will require longer mounting screws with a minimum pullout strength of 90kg (200lbs) (Not included in package).

POSITIONING

Seat user in wheelchair and establish correct posture with pelvic wheel belt. Position strap guides between neck and shoulder joint, approximately at shoulder height (adjust up/down for pressure distribution) Determine best strap guide orientation based on the user's needs.

INSTALLATION

Strap Guide only

5/32" (4mm)



13 NM MAX
(115 lbf-in)

INSTALLATION

with Mounting Adapter

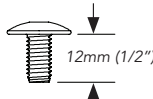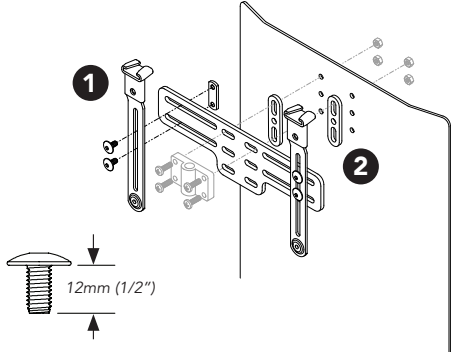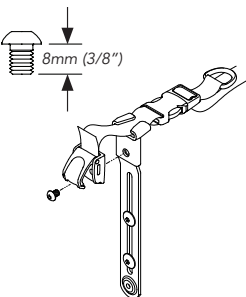
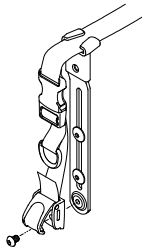
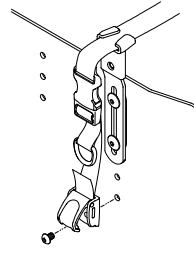
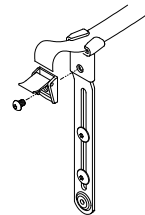
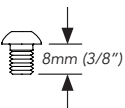
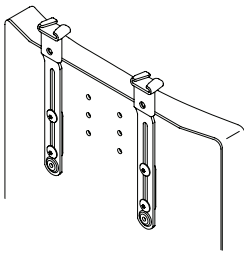
1. Attach strap guides to mounting adapter using 12mm (1/2") screws and nut plates.
2. Attach mounting adapter to the seat back using the headrest receiver and its screws, or other appropriate mounting hardware. (Not included in package)

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