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en These instructions provide important information for the safe use and maintenance
of the Bodypoint Evoflex™ Pelvic Support Belts. Give these instructions to the user or
their caregiver and review them to ensure that they are understood.

⚠️ WARNING! Product should be installed and fitted by a qualified rehab
technician.

USE:

⚠️ WARNING! This product should only be used for positioning a person in a
wheelchair. It is NOT intended for use as a transportation safety device, as a
personal restraint device, or in any other application where its failure could
result in injury. Misuse of this product is unauthorized and unsafe.

⚠️ WARNING! This pelvic stabilizer must be worn tightly fitted across the lower
pelvis or thighs at all times. A loose belt can allow the user to slip down and
create a risk of strangulation. Have your seating specialist demonstrate its proper
adjustment and use.

⚠️ WARNING! Accidental release of this pelvic stabilizer can allow the user to
slip down or fall from the wheelchair. If the user's movements or cognitive
abilities could lead to accidental release, a caregiver must be present at all times
during its use. Ensure that all caregivers know how to unfasten the product.
Failure to do so may delay release in an emergency.

⚠️ WARNING! As with any new seating support, this product may change the
way a person sits. Users must continue to practice regular pressure relief
activities and skin integrity checks, not only where this product contacts the user,
but also in primary pressure-bearing areas such as the sacrum, legs, and
buttocks. If increased skin redness or irritation occurs, discontinue use and
consult your physician or seating specialist. Failure to do so may result in serious
injury, such as pressure ulcers.

MAINTENANCE: Check periodically for signs of wear in the stitching, webbing,
and pads. If significant wear is found, contact your supplier for qualified repair
or replacement by Bodypoint. Under no circumstance should this product be
altered or repaired by unqualified persons—health and safety depend on it!

🧼🌟🔄 CLEANING: Machine wash, warm, 40°C (100°F). Do not bleach.
Tumble dry, low temperature, or drip dry. Do not iron. (Placing the product
inside a cloth bag during washing helps prevent scratches to the product and the
machine.)

WARRANTY: This product carries a limited lifetime warranty against defects in
workmanship and materials arising under normal use by the original consumer.
Contact your supplier or Bodypoint for warranty claims.

For more information on Bodypoint products, and a list of distributors outside the
USA go to www.bodypoint.com.

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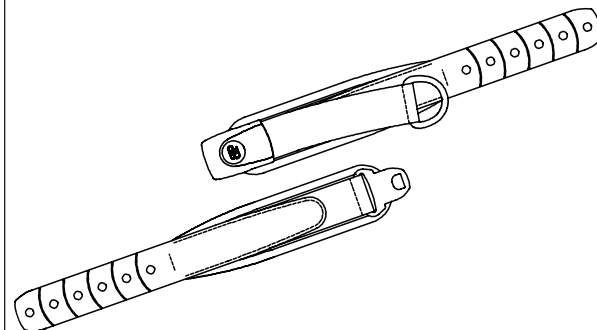
en

Evoflex™ Pelvic
Stabilizer Installation
and User's Instructions

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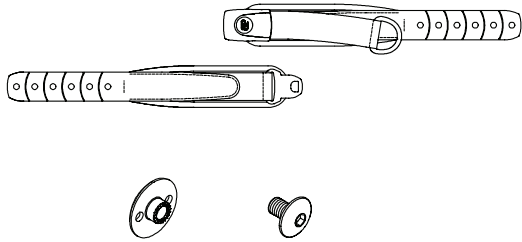
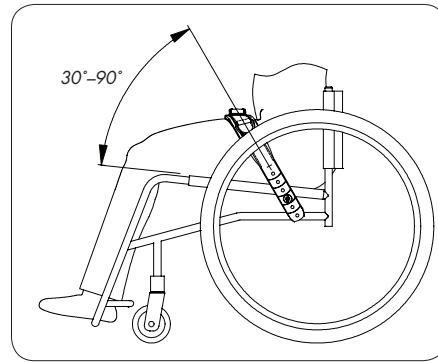
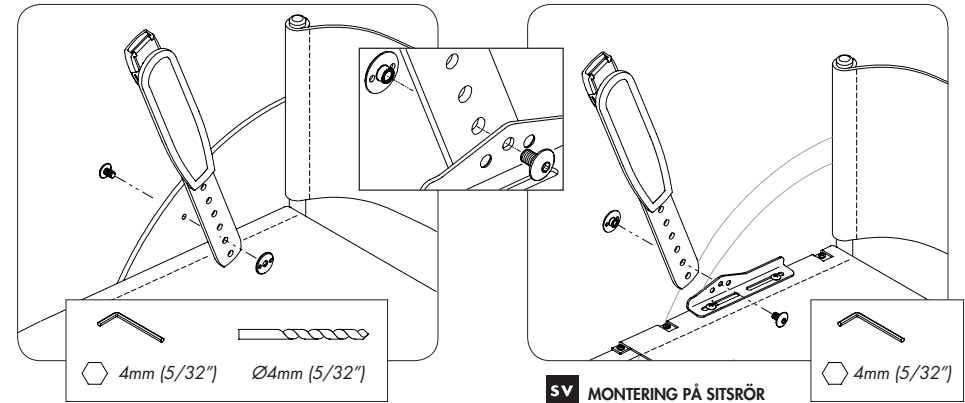
en BELT POSITION

The recommended belt angle is 30°-90° from the seat, accord-
ing to the positioning needs of the user. Determine the best posi-
tion for the belt and, with the adjustment strap fully tightened,
locate the desired anchor points, then select from the mounting
methods shown.

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en SIDE GUARD MOUNTING

Drill 6mm (1/4") hole or use existing hole/slot to attach
belt using included fasteners.

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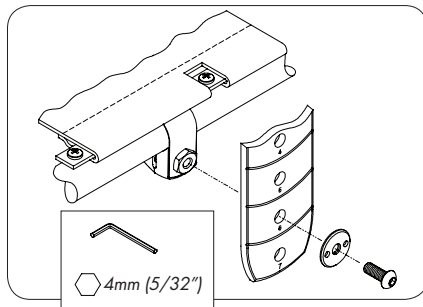
en SEAT TUBE MOUNTING

(Requires Art. No. HW312-KIT)
Attach mounting bracket to seat tube. Attach belt using
included fasteners through middle hole of HW312
mounting bracket.

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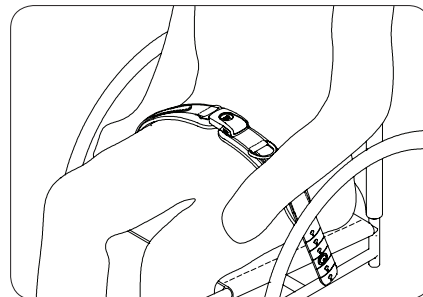
en BAND CLAMP MOUNTING

(Requires Art. No. HW320) Attach band clamp to seat
tube. Attach belt using the M6 X 12 screw included with
the HW320 band clamp.

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en ADJUSTMENT

When properly adjusted and the belt tightened, it should fit
snug so that the user's pelvis is secure. Tighten the D-ring pull
until all slack is removed and the pads under the buckle are
touching. If length adjustment is necessary to achieve a good
fit or to remove the product for cleaning, this can be done at
the end straps.

SAFETY CHECK

When properly positioned in wheelchair, have user lean
forward and side to side to check fit. Check for:

1. Normal operation of buckle and adjustment straps.
2. Comfort Look for areas of irritation.
3. Position If too high or too low, adjust anchor points.
4. Interference with other devices Relocate anchor points as
necessary.
5. Move chair through full range of motion, including tilt,
folding, and rolling. Check for any interference.