
Aeromesh® Rapid-Dry Bath Belt, One-Piece

INSTALLATION AND USER'S INSTRUCTIONS

ART NOS. BB111 & BB112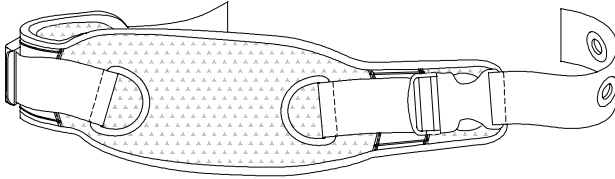

These instructions provide important information for the safe use and maintenance of all one-piece Bodypoint Aeromesh® Rapid-Dry Bath Belts. Give these instructions to the user or their caregiver and review them to ensure that they are understood.

⚠ WARNING! Because of the risk of choking, it is dangerous to use a chest belt without stabilizing the pelvis – always use with a properly fitted knee or thigh belt.

Optional Bodypoint belt mounting hardware may be used for mounting to wheelchairs and seating systems.

INTENDED USE:

⚠ WARNING! This product should only be used for positioning a person in a shower/commode chair. It is NOT intended for use as a transportation safety device, as a personal restraint device, or in any other application where its failure could result in injury. Misuse of this product is unauthorized and unsafe.

⚠ WARNING! This positioning belt must be properly fitted to support the user's body without causing injury. Have your seating specialist demonstrate its proper adjustment and use.

- If it is too tight across the shoulders and chest, it can restrict respiration and increase pressure.
- If it is too loose, it can allow the user to slip down and may create a risk of strangulation.
- Accidental release of this product can allow the user to fall forward or fall from the shower/commode chair.
- A user's inability to self-release can be hazardous if the user slips down or is trapped in the shower/commode chair in an emergency.

If the user's physical or cognitive abilities could prevent them from safely operating this product,

a caregiver must be present at all times during its use. Ensure that all caregivers know how to correctly adjust and unfasten the product.

⚠ WARNING! If a serious incident occurs related to the use of this product, it should be reported to the manufacturer (Bodypoint, Inc.) and the local Competent Authority.

PERIODIC SAFETY AND PERFORMANCE

CHECKS: To ensure user safety, this product must be checked periodically for function and signs of wear. If the product does not function correctly or if significant wear is found in the buckles, mounting points, webbing, padding, or stitches, stop using it and contact your supplier for qualified repair or replacement by Bodypoint. Under no circumstance should this product be altered or repaired by unqualified persons — health and safety depend on it!

🧺🧼🔄🧺 REGULAR CLEANING: Machine wash, hot, 60°C (140°F) using laundry detergent. Tumble dry, or drip dry. Do not iron.

SCRAP/DISPOSAL: Product is made of materials which can be disposed of safely without special precautions at the end of its useful life.

WARRANTY: This product carries a limited-lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

For more information on Bodypoint products, and a list of distributors outside the USA go to www.bodypoint.com

 **Bodypoint®**



Bodypoint Europe BV
Kerkstraat 29
7396 PD Terwolde
The Netherlands

MD Medical Device

558 First Ave. S., Suite 300 | Seattle, WA 98104 | 206.405.4555 | 800.547.5716 | www.bodypoint.com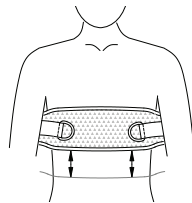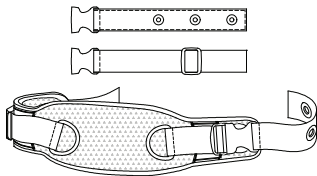


PACKAGE CONTENTS

Package may contain one or more of the following:

- Aeromesh® Rapid-Dry Bath Belt
- Mounting straps with grommets, or
- Mounting straps with slide adjuster
- Optional: Frame Clamps (Art. No. HW310), Band Clamps (Art. No. HW320), Knob (Art. No. FA812-BP)

POSITIONING

Seat user in shower/commode chair and establish correct posture. Determine desired position for the belt. When choosing position, consider interference with other devices. To reduce the risk of strangulation and trunk instability, the belt should be neither too high nor too low.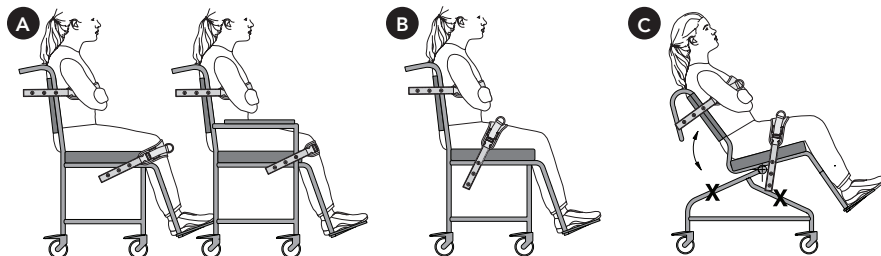

MOUNTING SUGGESTIONS


A At Chest, At Knees (Preferred)

Taking into account the user's needs, find a suitable mounting location on the wheelchair that allows the belt to be mounted at one of the preferred positions shown.

B Across Thighs (Optional)

C Tilting Chairs

Belt must be mounted to the upper frame, above the pivot point. Mounting the belt below the pivot point can cause interference or affect belt tension.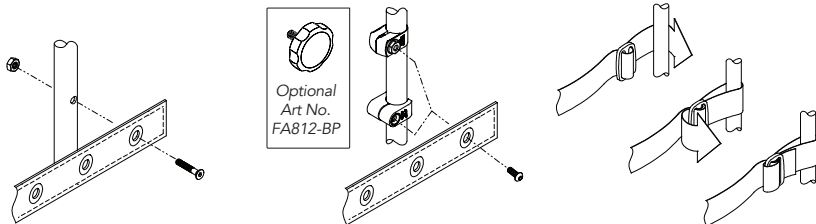

INSTALLATION


Direct Screw Mounting

If mounting directly to chair frame or solid seats/back, use 6mm (1/4-20) Flat head or Button head screws. Mounting screws must have a minimum pullout strength of 90kg (200lbs). Not included in package.

With Frame Clamps or Band Clamps

Attach clamps to shower/commode chair in desired locations and tighten in place. Insert second screw through grommet on belt and into the clamp. Tighten securely.

Mounting with Slide Adjuster

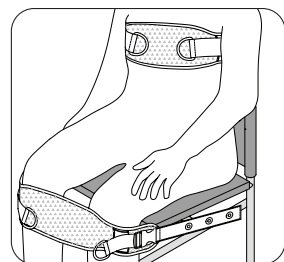ADJUSTMENT

With the user correctly seated, check the fit of the belt – it should be snug so the user is secure. Small adjustments in fit are done using the D-ring tightening straps. Large adjustments in fit are done using the mounting straps.

SAFETY CHECK

When properly positioned in chair, have user lean forward and side-to-side to check fit. Check for:

1. Normal operation of buckle and adjustment straps.
2. Comfort - Look for areas of irritation.
3. Position - If too high or too low, adjust mounting points.
4. Interference with other devices - Relocate mounting points as necessary.
5. Move chair through full range of motion, including tilt, folding, and rolling. Check for any interference.

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