


sales@bodypoint.con

## ARTICLE NUMBER TAG

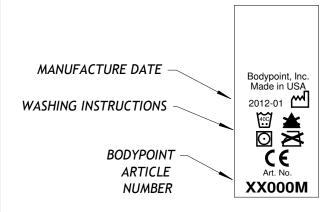

CLEANING: Machine wash, warm, 40°C (100°F). Do not bleach. Tumble dry, low temperature, or drip dry. Do not iron. (Placing the product inside a cloth bag during washing helps prevent scratches to the product and the machine.)

## SHOWER BELT LABEL



REGULAR CLEANING: Machine wash, hot, 60°C (140°F) using laundry detergent. Tumble dry, or drip dry. Do not iron.

SPECIAL CLEANING: For extra sanitizing, the belt can be machine washed, very hot, 70°C (160°F) with bleach.

	description				
SEWN PRODUCT WASHING INSTRUCTIONS					s
	drawing no.	date originated	drawn by	rev.	
	D11-0908-30	9/8/2011	D. HAGAN		$B \mid$

Bodypoint 558 1st Ave. South Seattle, WA 98104 USA tel (206)405-4555 fax (206)405-4556 www.bodypoint.com